
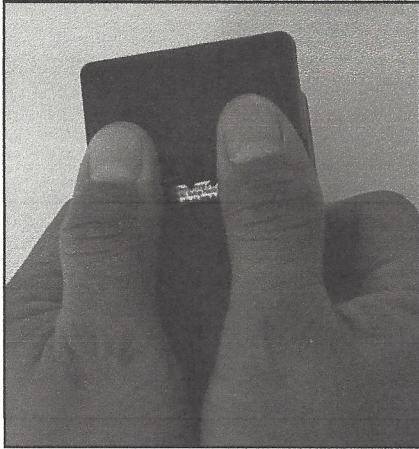


## Remote Control Information:

Your remote control operates with 2 “AA” batteries (included with this product). When the remote control is new, the latch on the battery door may be a bit stiff and difficult to open. It should loosen up and be easier to open after it has been opened and closed a few

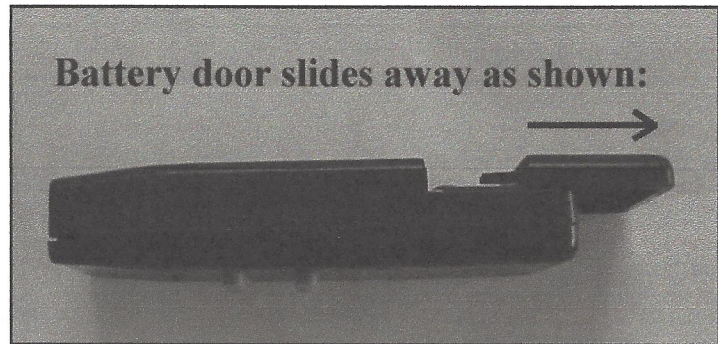

### Follow these steps to open the battery door:

1. Hold remote control so you are looking at the back side, battery door on the top.
2. Place your thumbs on the battery door and push away.

### A note on storing your remote control:

This remote control is designed to operate in a low-power standby mode when buttons are not pressed. To increase battery life, be sure to keep the buttons free of objects and surfaces that may push on the buttons when the remote is not in use. If you store the remote control in the carrying case, make sure that the buttons are not pressed on in the case.