

Tripod setup instructions.

1. Remove the mounting plate from the tripod by moving the thumb clamp away from the tripod as shown in figure 1 and lifting mounting plate away from tripod base.

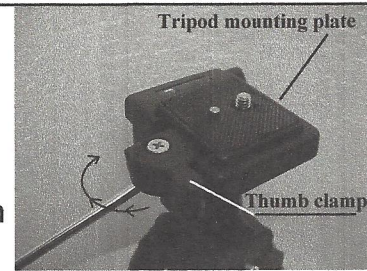


Figure 1

2. Align the mounting plate threaded screw with the light-bar threaded hole and the mounting plate alignment pin with the light-bar alignment hole located next to the light-bar's threaded insert (Figure 2).

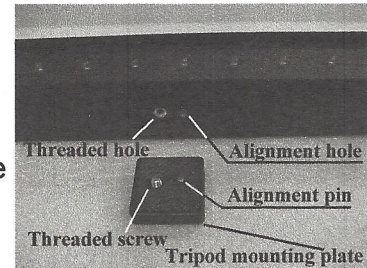


Figure 2

3. Tighten the threaded screw using your thumb and forefinger (Figure 3)

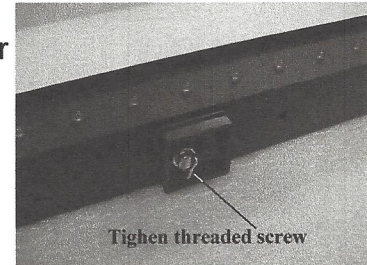


Figure 3

4. Mount the light-bar to the tripod by reversing the process used in step 1 (Figure 4).

5. When disassembling the light-bar from the tripod for storage or transport, it is not necessary to remove the mounting plate from tripod. This will speed re-assembly. Simply release thumb clamp and lift off light-bar

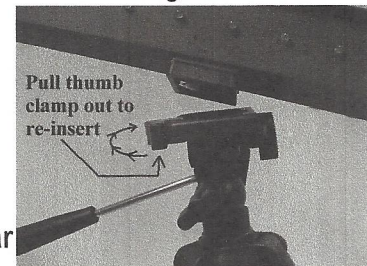


Figure 4

NeuroTek Corporation
12100 W. 52nd Ave. #116
Wheat Ridge CO 80033

303-420-8680
303-422-9440 fax
www.neurotekcorp.com