

Tripod setup instructions

1. Remove the mounting plate from the tripod by moving the thumb clamp away from the tripod as shown in figure 1 and lifting mounting plate away from tripod base.

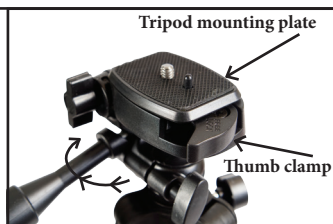


Figure 1

2. Align the mounting plate threaded screw with the light-bar threaded hole (Figure 2).

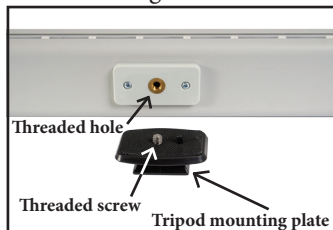


Figure 2

3. Tighten the threaded screw using your thumb and forefinger (Figure 3)

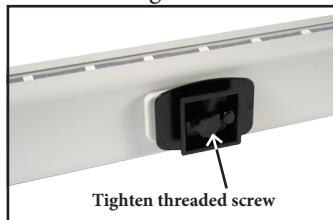


Figure 3

4. Mount the light-bar to the tripod by reversing the process used in step 1 (Figure 4).

5. When disassembling the light-bar from the tripod for storage or transport, it is not necessary to remove the mounting plate from tripod. This will speed reassembly. Simply release thumb clamp and lift off light-bar.



Figure 4